

**San Francisco General Hospital & Trauma Center**  
**Point of Care Testing**  
**HEMOCUE Quality Control Log**

R&D HEMOCUE CONTROL: Level 3 (HIGH)

LOCATION: \_\_\_\_\_

LOT# GH05243

EXPIRATION DATE: **08/10/2024**

DATE OPENED: \_\_\_\_\_ OPEN VIAL EXP.: \_\_\_\_\_ (=30 DAYS AFTER OPEN)

DATE OPENED: \_\_\_\_\_ OPEN VIAL EXP.: \_\_\_\_\_ (=30 DAYS AFTER OPEN)

1. Run a control test every day of and prior to patient testing.
2. Verify that the lot# on the **control sample** (not the box) is the same as on this form.
3. Mix control sample **GENTLY & THOROUGHLY** and run as you would a patient sample.
4. Record result on this form. If result falls outside range, repeat. If still out, do not report patient results.
5. If the control results falls between the range, report patient results.
6. QC solutions are good for up to **30 days once OPENED** and stored either at Room Temp or Refrigerated.
7. For help: contact POCT Services

✓ = Daily	DATE	HGB	TEST	✓ = Daily	DATE	HGB	TEST
Maintenance	MEAN	<b>15.5</b>	PERFORMED	Maintenance	MEAN	<b>15.5</b>	PERFORMED
Done	RANGE	<b>14.3-16.7</b>	BY	Done	RANGE	<b>14.3-16.7</b>	BY
	S				S		
	M				M		
	T				T		
	W				W		
	Th				Th		
	F				F		
	S				S		
	S				S		
	M				M		
	T				T		
	W				W		
	Th				Th		
	F				F		
	S				S		

Comments:

---



---



---